# Cavitron® Steri-Mate™ Light

DENTSPLY

Lighted Sleeve for Steri-Mate® Handpieces

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## INTRODUCTION

The Steri-Mate™ Light is a steam autoclave sterilizable multi-use disposable accessory for the 30K Steri-Mate® sterilizable handpiece. The light easily slides on the Steri-Mate handpiece for use with all 30K Cavitron® Inserts. The light provides optimal light intensity without requiring an external power source and operates at all power settings including the low power Cavitron Blue Zone.

## INDICATIONS FOR USE

- Provides illumination to enhance visibility in the clinical working area during ultrasonic scaling procedures with the Steri-Mate handpiece.

- When the Steri-Mate™ Light is installed on the Steri-Mate handpiece, the soft tapered grip provides multiple pinch points, to enhance ergonomics and improve clinician comfort.
- The Steri-Mate™ Light is sterilizable which reduces the risk of cross contamination.

## CONTRAINDICATIONS

- Ultrasonic systems should not be used for restorative dental procedures involving the condensation of amalgams.

## WARNINGS

- Persons fitted with cardiac pacemakers,

defibrillators and other active implanted medical devices, have been cautioned that some types of electronic equipment might interfere with the operation of the device. Although no instance of interference has ever been reported to DENTSPLY, we recommend that the handpiece and cables be kept at least 6 to 9 inches (15 to 23 cm) away from any device and their leads during use.

- There are a variety of pacemakers and other medically implanted devices on the market. Clinicians should contact the device manufacturer or the patient's physician for detailed information about the device.

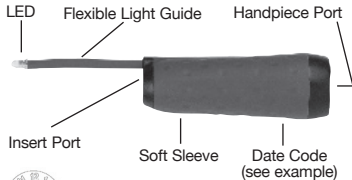
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- It is suggested that prior to beginning treatment, patients should rinse with a known antimicrobial such as Chlorhexidine Gluconate 0.12%. Rinsing with an antimicrobial reduces the chance of infection and reduces the number of microorganisms the patient might release in the form of aerosols during treatment.

- DO NOT USE if the LED light guide insulation is torn or damaged exposing electrical conductors. Discard the Lighted Sleeve immediately.

## PRECAUTIONS

- Sterilize before use. Follow the infection control procedures carefully.

## STERI-MATE™ LIGHT DESCRIPTION

Example Date Code  
(January 2007)

## INSTALLATION INSTRUCTIONS

1. Sterilize before installation, following the Infection Control Procedures.
2. Place the handpiece port of the sleeve over the top of the Handpiece and slide the sleeve on the Steri-Mate handpiece until fully seated.

3. Lubricate the o-ring on the insert with water. Fill the entire Steri-Mate handpiece with water. Gently twist the insert down into the Steri-Mate handpiece until the insert is fully seated in place. **DO NOT FORCE.**

## DIRECTIONS FOR USE

1. Rotate the sleeve to direct the light to the desired position.
2. The flexible light guide is adjustable to improve visibility of the clinical working area.
3. The Steri-Mate™ Light is now ready for use. To activate the LED, depress the foot control.

## INFECTION CONTROL PROCEDURES

### WARNINGS

- The Steri-Mate™ Light is not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.
- The Lighted Sleeve must be disassembled from the Handpiece prior to autoclaving. Not following this procedure will result in defective cosmetic appearance.
- The colorant used on the sleeve may bleed and discolor during autoclaving. This will not affect performance.
- Allow the Lighted Sleeve and the Handpiece to cool after autoclaving for proper assembly.

### LIMITATIONS ON PROCESSING

- The Steri-Mate™ Light is designed to withstand a minimum of 100 autoclave cycles. When illumination of the LED begins to diminish or if the insulation is torn or damaged, replace the Lighted Sleeve.
- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the materials may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
- Do not immerse the Lighted Sleeve in an ultrasonic bath.

## INSTRUCTIONS

### Point of use:

Remove excess soil with disposable cloth or paper.

### Containment and Transportation:

Protect the LED from damage. Reprocess as soon as is reasonably practical following use. Lighted Sleeves left standing wet may stain.

### Cleaning: manual

- Disassemble the Lighted Sleeve from the Handpiece. Rinse under running water to remove any gross debris.
- Wipe Handpiece and Lighted Sleeve with a cloth saturated with multi-purpose cleaner and rinse with running water. Discard used cloth. Dry with a clean cloth.

### Disinfection:

Disinfection of the Steri-Mate™ Light is not necessary prior to steam autoclave sterilization.

### Packaging:

With Lighted Sleeve and Handpiece disassembled, place instruments into a paper or paper/plastic steam sterilization pouch. The instruments may be loaded into dedicated instrument trays.

### Sterilization:

Use a steam autoclave. Place bagged instruments into the steam autoclave, paper side up when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 273°F/134°C to 279°F/137°C and pressure of 31psi/216kPa for 3 to 12 minutes.

Alternate method: Place non-bagged instruments into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature and pressure of 273°F/134°C to 279°F/137°C and pressure of 31psi/216kPa for 3 to 4 minutes. The instruments should be used immediately after drying.

#### Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes. Allow instruments to cool before use.

#### Maintenance, Inspection and Testing:

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded instruments.

#### Storage:

To maintain sterility, instruments should remain bagged until ready for use.

## MANUFACTURER CONTACT

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Professional Representative.

## WARRANTY

The Cavitron® Steri-Mate™ Light manufactured by DENTSPLY® Professional is warranted against

defects arising from faulty materials or workmanship for thirty days from the date of purchase. This warranty extends to product purchased from an authorized DENTSPLY distributor and only to the original purchaser and is valid only for Lights purchased in the United States (U.S.) and for those countries without specific warranty requirements. For Lights purchased outside the U.S. in countries with specific warranty requirements, those applicable country warranties will apply. This warranty is subject to the following conditions:

- 1) Lights must not be subjected to abuse or improper use or application;
- 2) Lights must not be subjected to abuse or improper cleaning, disinfection, or sterilization procedures;

- 3) Using Steri-Mate™ Lights on handpieces other than Cavitron Steri-Mate handpieces will void this warranty.

There are no warranties, expressed or implied beyond the description on the face hereof. DENTSPLY neither assumes, nor authorizes any person to assume for it, any other liability for incidental and consequential damages resulting from the use of the insert.

To receive reimbursement for any defective Cavitron Steri-Mate™ Light, you must send the defective product within the warranty period to the dealer or manufacturer with specification of the defect and receipt of purchase.

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DENTSPLY DeTrey GmbH